


Roger ON

Using the Roger ON radio aid helps the student overcome the problem of unwanted background noise, sound reverberation and the distance between the speaker and the student with hearing impairment.

The system consists of a transmitter (the Roger ON) and the receivers. The transmitter relays your voice directly into the receivers.

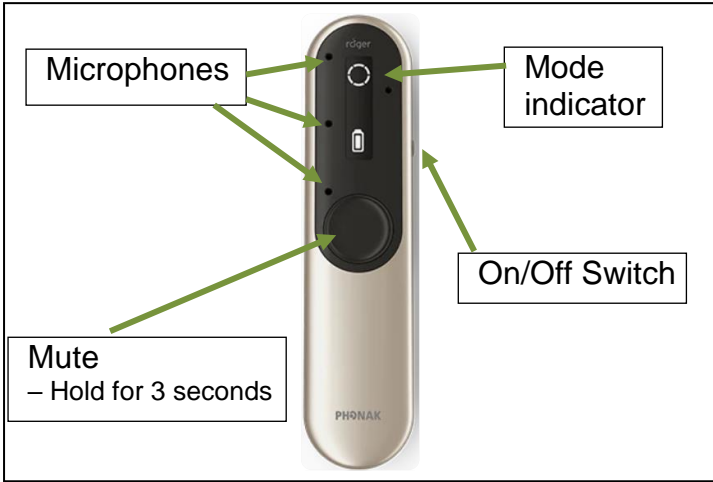

How to wear the transmitter.

Ensure lanyards, jewellery, scarves etc. are not knocking against the transmitter.



Adjust the Lanyard - Make sure that the microphones are a hand span from your mouth

How to use

- Ensure that the On is turned on by pressing and holding down the side on/off button
- Place the transmitter round your neck using the lanyard or clip on a jacket or pocket.
- Make sure that the microphones are a hand span away from your mouth and not covered.
- Mute the radio aid if you are not talking directly to the student. Push the mute button once. Push the button again to unmute it.
- Conference/Small group mode: When the pupil is working in a small group, place the Roger Pen horizontally on the centre of the table. The student will be able to hear the whole group.
- To turn off press and hold down the on off button.
- Charge overnight using the direct connection or the dock.



Related Sheets:

Hearing aids

Radio aids



Other resources

<https://www.phonak.com/en-uk/hearing-devices/microphones/roger-on>

www.ndcs.org.uk