



Phonak PartnerMic

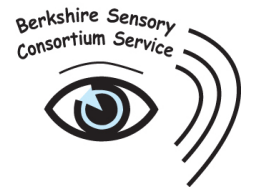

The **Phonak PartnerMic** is a wireless microphone that streams a speaker's voice directly to a Child or Young Person's (CYP) hearing aids, improving clarity in the classroom and other education settings

- ✓ Streams up to **25 metres (80 ft)**
- ✓ Up to **6 hours** of use per charge
- ✓ Fully charges in about **2 hours (USB-C)**
- ✓ Volume adjustable via app or hearing aid

Pairing (usually only needed to be done once)

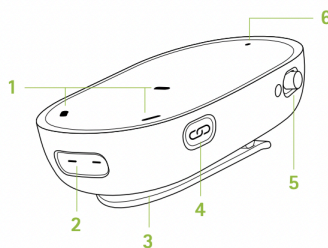

- 1 Microphone openings
- 2 Charging input
- 3 Clip
- 4 Pairing button
- 5 Power switch On/Off
- 6 Indicator light

1. Turn on the **hearing aids** and **PartnerMic**
2. Hold the PartnerMic within **10 cm (4 inches)** of the hearing aid
3. Press the **pairing button**
4. Light will **blink blue** during pairing and turn **solid green** when connected
 - A beep may be heard in the hearing aids

Daily Use Tips

- Clip vertically about **20 cm (8 inches)** below the speaker's mouth
- Obstacles like walls may reduce range

Troubleshooting

If no sound:

- Ensure device is **charged and turned on**
- Check it is **within range**
- A **solid or flashing blue light** means it is searching for the hearing aid