


Roger Touchscreen

Using the Roger Touchscreen radio aid helps the pupil overcome the problem of unwanted background noise, sound reverberation and the distance between the speaker and the child with hearing impairment.

The system consists of a transmitter (the Roger Touchscreen) and the receivers. The transmitter relays your voice directly into the receivers.

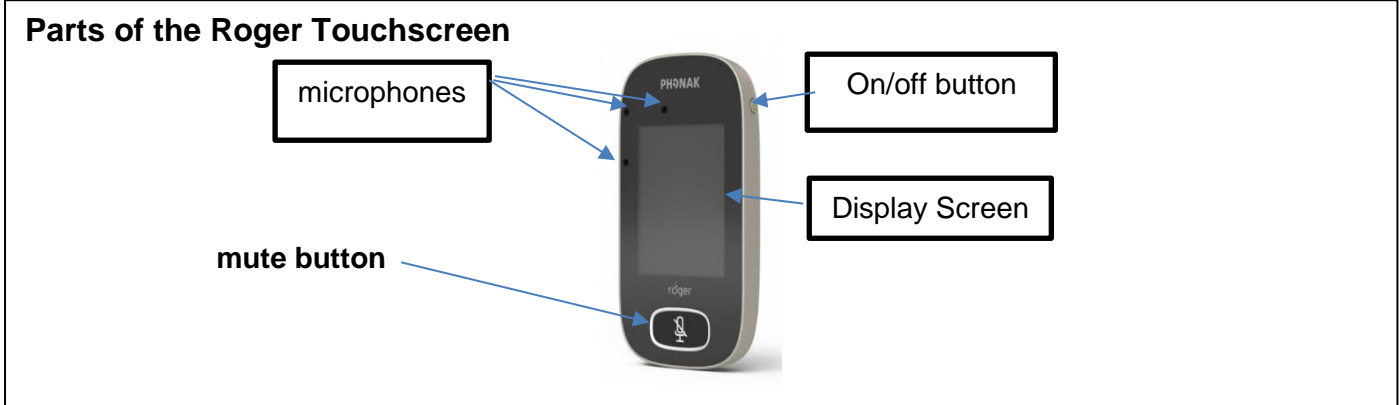

How to use

- Ensure that the touchscreen is turned on by pressing and holding down the side on/off button for approximately for 3 seconds.
- Place the transmitter round your neck using the lanyard.
- Make sure that the microphones are a handspan away from your mouth
- Ensure that the microphones are not covered. Check your badge or clothing is not covering or rubbing against the mics.
- Mute the radio aid if you are not talking directly to the pupil. Push the button on the bottom of the screen once. It will start flashing red. Push the button again to unmute it.
- Conference/Small group mode: When the pupil is working in a small group, place the Roger Touchscreen flat on the centre of the table. The pupil will be able to hear all peers.



- To turn off press and hold down the side on/off button for approximately 3 seconds. The screen will go grey in colour and display 'Power off?'. Press 'Yes' to confirm.

Other resources

www.phonak.com

www.ndcs.org.uk

Related Sheets

Hearing aids

Radio aids